



How to Use the QuickScan 650 Scanner

1. Insert batteries prior to use.
2. Continuously press and hold the red button to turn the scanner on – it will beep and the LCD screen will light up when the sensors are activated. If the scanner has been used before, it will display the number of the last microchip read.
 - Still pressing the red button, hold the scanner with the red “scanning” sensor pointed down toward the pet. Slowly scan the area between the shoulder blades in a zigzag motion.
 - If no microchip is detected between the shoulder blades, scan a broader area, including the front legs down to the elbows and the neck.
3. When a microchip is detected, the scanner will beep and the screen will display the pet’s microchip number.
4. To help reunite a found pet with its owner, go to www.aaha.org/petmicrochiplookup or call AKC Reunite at 800-252-7894.

The scanner will automatically turn itself off approximately 30 seconds after you stop pressing the red button.

For future reference: if the scanner is working slowly, replace the 9 volt battery.

For best results, remove metal collars before scanning. Computers, some automobile key FOBS, metal tables and even fluorescent lighting can interfere with effective scanning.



This scanner comes with a twelve-month manufacturer's warranty. Please do NOT write on, mark on or otherwise damage or deface this scanner as this may void the warranty.

⚠ WARNING: products with exposed solder may contain lead, a chemical known to the State of California to cause birth defects and other reproductive harm. Please wash hands after handling internal components and motherboards and avoid inhalation of fumes if heating the solder.

HOW TO SCAN FOR A MICROCHIP

FIRST CHECK BETWEEN THE SHOULDERS:

1. Using a Universal Scanner, start scanning slowly between the pet's shoulder blades, going side-to-side in an "S" shaped pattern.
2. If no chip is detected, **rotate the scanner 90 degrees** and scan again between the pet's shoulder blades.
3. Then, start at the pet's head, and slowly scan side-to-side all the way to the pet's tail, **making several passes** over each area.
4. When you find a microchip, always check where it is enrolled on petmicrochiplookup.com!

IF NO CHIP IS DETECTED:

1. Make sure you have **fresh batteries**!
(We recommend Duracell® or Energizer®)
2. **Slowly scan** down the sides and behind the pet's legs, across its neck, and under its chest.

 = Antenna Location That Reads Microchip

PROSCAN 700



QUICKSCAN 650



WANDSCAN 900 REACH



WANDSCAN 900 PRO



It can take up to **2 minutes** to detect a pet's microchip



Always check the chip at
aaha.org/petmicrochiplookup

Brought to you by AAHA



www.akcreunite.org

800-252-7894

FIND US ON:   